

(continued) Appendix B. Repairer PMCS

4110-01-159-6922
 Refrigerator, Mechanical, Blood Bank, Model 139875

[M-Monthly, Q-Quarterly, S-Semiannually, and A-Annually]

ITEM NO	INTERVAL	ITEM TO BE INSPECTED AND PROCEDURE	IS NOT MISSION CAPABLE IF:
1	S	<p>Refrigerator</p> <p>a. Ensure that a copy of the manufacturer's manual is on hand.</p> <p>b. Inspect the refrigerator for obvious signs of damage such as cracks, dents, leaks or broken components.</p>	<p>The power cord is cracked or frayed, wires are not covered by the cord insulation, or the damage prevents the refrigerator from operating.</p>
2	A	<p>Installation of the Refrigerator</p> <p>Verify that the refrigerator has been installed according the Operator Preventive Maintenance Checks and Services.</p>	
3	A	<p>Maintaining Refrigerator</p> <p>a. Verify that the refrigerator has been maintained according to the Operator Preventive Maintenance Checks and Services.</p> <p>b. Verify electrical safety.</p>	<p>The refrigerator fails any of the electrical safety tests.</p>