

FIXATION SET, ORTHOPEDIC

MANUFACTURER: STRYKER/HOWMEDICA

Manufacturer Model# 4920-9-952

Manufacturer's PHONE: (201) 831-5726 FAX: (201) 831-6553

Manufacturer's Website: <http://www.trauma.stryker.com>

Item Web Link:

<http://www.trauma.stryker.com/products/hoffmann2.php?tab=&vis=&loc='>>

AAC 'L' ITEM NSN 6515-01-546-0559

UA's: 267B , N302 , N308 , N542 , N742 , N783


USAMMA POC: Customer Relations Management 301-619-4301/4316 - DSN 343-4301/4316

UNIT PRICE: \$2,002.00

System Description: External fixation is a method of immobilizing bones to allow a fracture to heal. External fixation is accomplished by placing pins or screws into the bone on both sides of the fracture. The pins are then secured together outside the skin with clamps and rods. The clamps and rods are known as the "external frame." Some of the advantages of external fixation are that it is quickly and easily applied. The risk of infection at the site of the fracture is minimal, but there is a risk of infection where the pins are inserted from the skin into the bone. The Forward Deployed Field Pack is for temporary stabilization and transportation only.

Features:

- Modular frames which can be tailored according to surgeon's needs, allow for true independent pin placement
- The "snap click" mechanism makes it a quick and easy system to apply with an endless versatility, ideal for temporary fixation or definitive treatment of extremely complex fractures, while respecting the soft tissues
- The independent pin placement and modular components allow for fracture reduction after the fixator is applied
- Additionally, the frame can be adapted according to the fracture biomechanics

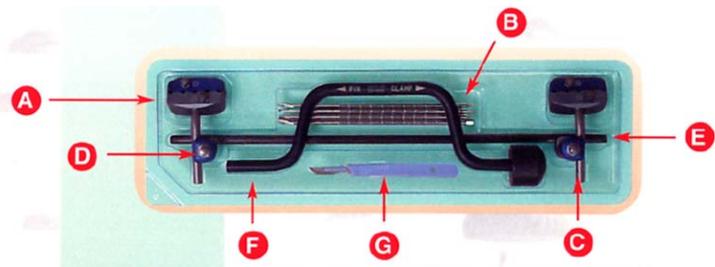

Component Listing	
	Description
A	Multi-Pin Clamp Assembly
B	5mm x 180mm Apex® SD/ST Half Pins
C	30° Angled Post, Stainless Steel
D	Rod-to-Rod Coupling 8/8mm
E	Carbon Connecting Rod 8mm x 400mm
F	Pin Driver/Wrench Assembly
G	#10 Scalpel

FIXATION SET ORTHOPEDIC
PART # 4920-9-952
NSN 6515015460559

**FIXATION SET, ORTHOPEDIC
PHYSICAL & ENVIRONMENTAL SPECIFICATIONS**

Item Dimensions	Height (Inches)	16.5
	Width (Inches)	7
	Length (Inches)	2
	Cube (Cubic Feet)	0.1337
	Weight	1.9 lbs.
Ship Dimensions	Height (Inches)	
	Width (Inches)	
	Length (Inches)	
	Cube (Cubic Feet)	
	Weight	
Storage Temp		
Operating Temp		
Transport Temp		
Ambient Temp		
Storage Humidity		
Operating Humidity		
Operating Altitude		
Amps		
Watts		
Power Requirements	N/A	
Notes		

FIXATION SET, ORTHOPEDIC

ACCESSORIES & CONSUMABLES - STARTUP

NSN	Notes	Nomenclature	PN	QTY	UI	Unit Price	Total Price	MFG/ DIST	Shelf Life Months	Refrig Req
6515015460559		Fixation Set, Orthopedic	4920-9-952	1	EA	\$2,002.00	\$2,002.00	Stryker/Howmedica	24	N
6515014631475		Pin Clamp Assembly w/No Post Ortho	4920-2-020	2	EA	Incl w/NSN 0559	Incl w/NSN 0559	Stryker/Howmedica	INDEF	N
6515014631497		Post Angle Fixation Ortho 30 Deg SS	4920-2-140	2	EA	Incl w/NSN 0559	Incl w/NSN 0559	Stryker/Howmedica	INDEF	N
6515014631498		Coupling Fixation Rod-to-Rod Ortho 8 / 8mm	4920-1-010	2	EA	Incl w/NSN 0559	Incl w/NSN 0559	Stryker/Howmedica	INDEF	N
6515015040434		Rod Carbon Connecting 400mm Long x 8mm Wide	5029-8-840	1	EA	Incl w/NSN 0559	Incl w/NSN 0559	Stryker/Howmedica	INDEF	N
6515015131261		Pin Bone 5mm Dia x 180 mm Long	5018-6-180	7	EA	Incl w/NSN 0559	Incl w/NSN 0559	Stryker/Howmedica	INDEF	N
No NSN Required		#10 Scalpel		1	EA	Incl w/NSN 0559	Incl w/NSN 0559	Stryker/Howmedica	INDEF	N
No NSN Required		Pin Driver/Wrench Assembly	5194-6-000	1	EA	Incl w/NSN 0559	Incl w/NSN 0559	Stryker/Howmedica	INDEF	N

FIXATION SET, ORTHOPEDIC
ADDITIONAL NOTES, WEB LINKS & TABLES

CLICK LINK BELOW FOR OPERATOR'S MANUAL

http://www.trauma.stryker.com/medias/pdf/hoffmann2_optech_50751001b3606.pdf