

Operator Preventive Maintenance Checks and Services

6525-01-384-9296

X-Ray Apparatus, Model LCROKS

[B-Before Operation, D-During Operation, A-After Operation, Q-Quarterly, and S-Semiannually]

ITEM NO	INTERVAL	ITEM TO BE INSPECTED AND PROCEDURE	IS NOT MISSION CAPABLE IF:
1	B, D, A	<p>X-Ray Apparatus</p> <p>a. Conduct an inventory to ensure that the items listed on the Equipment Parts and Accessories List are on hand.</p> <p>b. Inspect unit for damage, discoloration, or excessively worn components.</p> <p>c. Verify that the Verification/Certification sticker (DD Form 2163) has a current date.</p>	<p>Missing components prevent the use of the x-ray.</p> <p>Unserviceable components prevent the use of x-ray.</p> <p>The unit has not been verified within the last 12 months.</p>
2	B	<p>X-Ray Operational Test</p> <p>Conduct "Operator Maintenance" as directed by manufacturer's literature.</p> <p>a. Check control panel for nicks, scratches, and/or dents.</p> <p>b. Ensure proper seating of APR labels.</p> <p>c. Inspect unit for all warning labels, serial tags, UL, and CSA tags.</p>	<p>The labels are missing, unreadable, or outdated.</p>