

ROTATOR LABORATORY ROTATOR/SHAKER F/TEST CARDS EI TOC

BEACON BIOLOGICALS INC.

MODEL # 3010

Manufacturer's Phone: (561)-394-7825 FAX (561) 394 -7825 Ext. 8

Website: <http://www.beaconbiologicals.com/pages/413058/index.htm>

ACC "W"NSN ITEM 6640-01-172-1132

ACC "J"NSN ITEM 6640-01-498-3006

UA's: N303, N403, N503, N703

USAMMA POC: Materiel Acquisition Directorate, 301 619-7403, DSN 343-7403

UNIT PRICE: \$350.00

System Description: Rotator/Shaker for test cards, for use with the Daya Sypshilis Test System

ROTATION: The recommended speed of the rotator is 100 rpm, (98 - 102 rpm range). Below 98 and above 102 rpm's there is a tendency for the clumping of the antigen to be less intense in tests with undiluted specimens. This may result in some minimal reactions to be missed. In quantitation, rotation above 102 rpm's tends to produce a decrease in titer, approximately one dilution lower.



**ROTATOR LABORATORY ROTATOR/SHAKER
MODEL # 3010**

