

## TABLE, OPERATING

**MANUFACTURER: STERIS CORPORATION**

**Manufacturer Model# 2080 MIA**

**Manufacturer's PHONE: (440) 354-2600 FAX: (440) 639-4450**

**Manufacturer's Website: <http://www.steris.com>**

**AAC 'J' ITEM NSN 6530-01-353-9883**

**USAMMA POC: Customer Relationship Management 301-619-1288/4301 - DSN 343-1288/4301**

**UNIT PRICE: \$21,680.73**

**System Description:** The 2080MIA, (Manual Control, Image Amplification), is a manually operated pedestal-style operating table. Table has an articulating top section and adjustable foot and head sections for superior patient positioning with maximum convenience for the surgical team. Designed to accommodate patients weighing up to 300 pounds. Includes foot pedal operated conductive casters and locks designed to compensate for uneven floor surfaces.

**Features:**


Positions:

- Posturing for Imaging Intensification
- Total Hip Replacement
- Mastectomy
- Thoracic
- Kidney
- Lumbar Laminectomy
- Proctology
- Gall Bladder
- Abdominal
- Gynecology
- Cystoscopy


**NOTE: THIS TABLE IS STILL IN STOCK AT THE DEPOT BUT IS BEING REPLACED BY GETINGE USA INC. MODEL # 1118.01M0US**




**TABLE, OPERATING  
PART # 2080 MIA  
NSN 6530013539883**

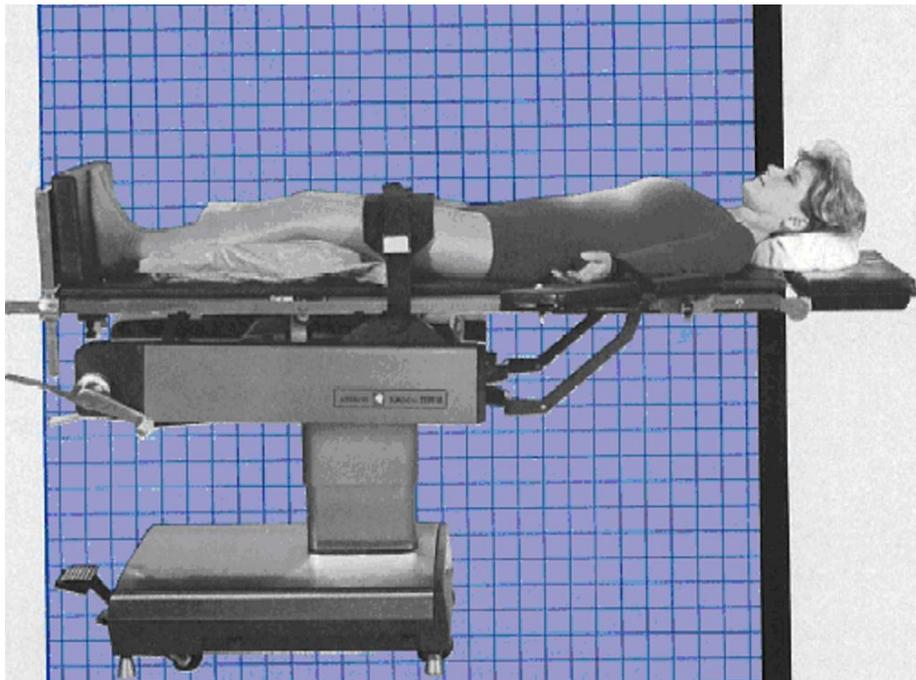


**POSTURING THE 2080 MIA SERIES TABLES PROVIDE AMPLE ACCESS FOR ANY FIXED OR MOBILE IMAGE INTENSIFICATION EQUIPMENT. PROCEDURES SUCH AS HEART CATHETERIZATION, PACEMAKER IMPLACEMENT, OR FOREIGN BODY LOCALIZATION ARE POSSIBLE ON ANY 2080 TABLE. THIS REDUCES THE NEED FOR A SPECIAL IMAGE INTENSIFICATION TABLE.**

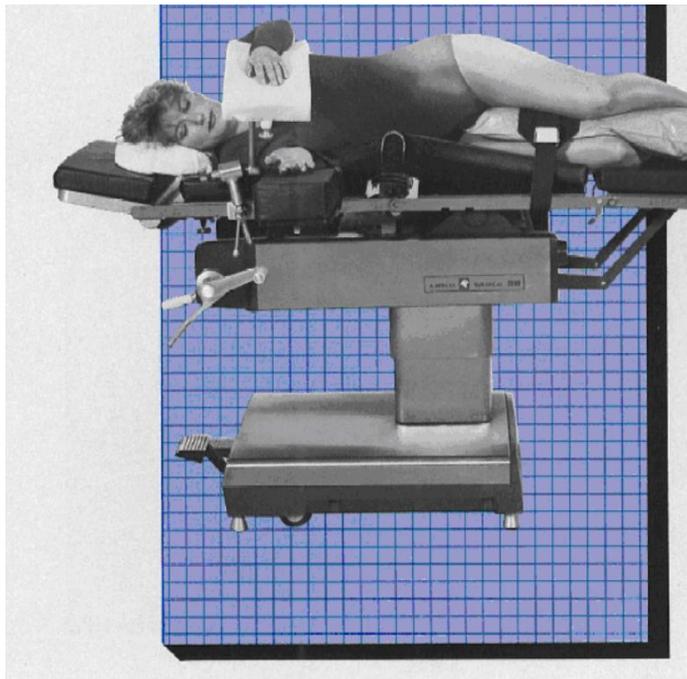


**TOTAL HIP REPLACEMENT - FOR THE LATERAL APPROACH TO A TOTAL HIP PROCEDURE, THE PATIENT IS POSITIONED WITH THE AFFECTED SIDE UPPERMOST. THE LOWER LEG SHOULD BE FLEXED AT A DEGREE SUFFICIENT TO MAINTAIN STABILITY OF THE TORSO; IT CAN BE SECURED BY MEANS OF A CONDUCTIVE BODY RESTRAINT STRAP PLACED OVER A SUITABLE PAD OR PILLOW. THE TORSO MAY BE FURTHER STABILIZED BY THE LATERAL SUPPORT OF A VACUUM EVACUATED MATTRESS.**

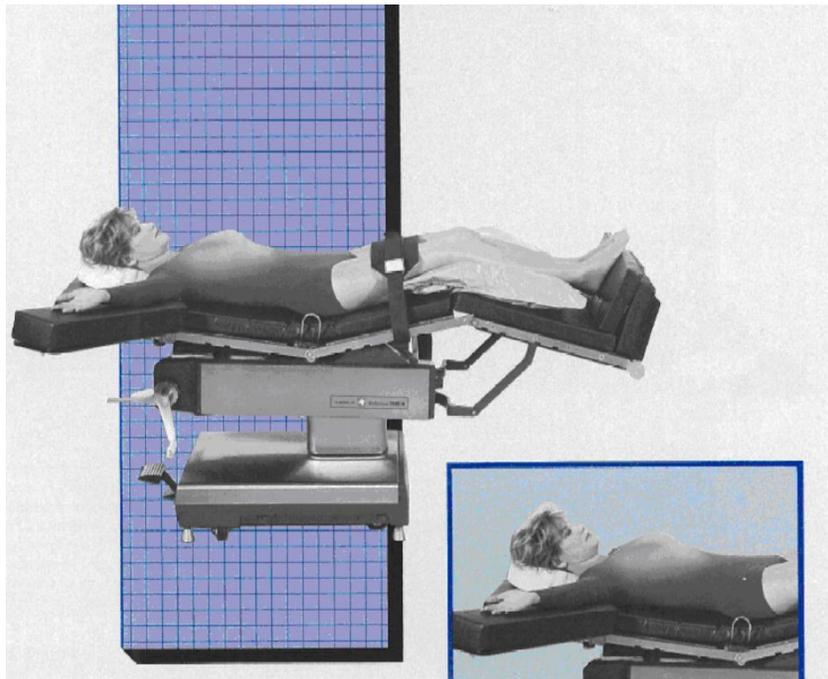


**MASTECTOMY THE CLEAN LINES OF AMSCO'S 2080 SURGICAL TABLE SERIES ARE ADVANTAGEOUS FOR SURGICAL PROCEDURES INCLUDING MASTECTOMY. ANESTHESIOLOGIST POSITIONS THE PATIENT IN A MODIFIED FOWLER POSTURE WITH SLIGHT LATERAL TILT AND HEAD SECTION LOWERED. ARM ON THE AFFECTED SIDE IS EXTENDED Laterally FOR EXCELLENT EXPOSURE TO MEDIAL AND LATERAL FLAPS.**

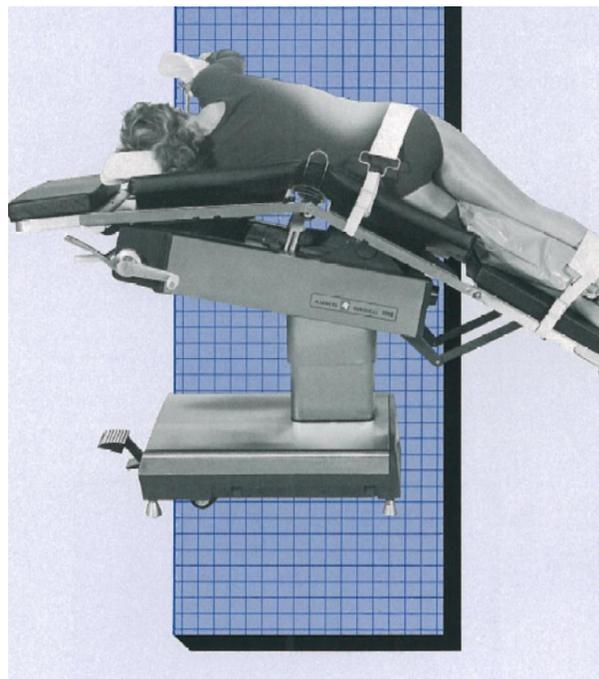


**THORACIC FOR RIGHT LUNG SURGERY THE OPERATING AREA IS EXTENDED, AND PRESSURE IS REDUCED ON THE DEPENDENT THORACIC CAGE BY POSITIONING THE SPACE BETWEEN THE ILIAC CREST AND THE GREATER TROCHANTER DIRECTLY OVER THE FLEX BETWEEN THE SEAT AND BACK SECTIONS. PATIENT'S UPPER ARM IS PLACED IN A KRAUS ARM SUPPORT.**

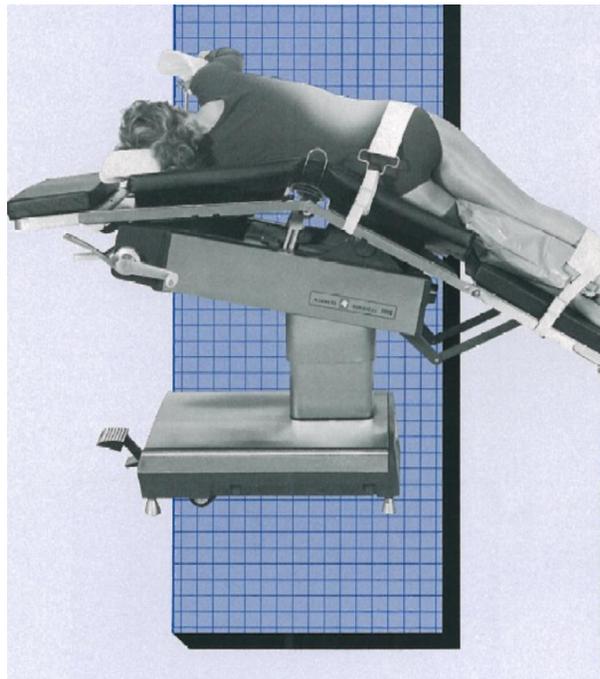


**KIDNEY PROCEDURES ARE ACCOMMODATED EASILY ON 2080 SERIES TABLES. PATIENT IS POSITIONED WITH ILIOCOSTAL SPACE DIRECTLY OVER THE FLEX BETWEEN THE SEAT AND BACK SECTIONS. LATERAL BODY SUPPORTS ARE POSITIONED BELOW THE LUMBAR VERTEBRAE AND TABLE IS FLEXED. THE ADHESIVE TAPE HOLDER FURTHER STABILIZES THE PATIENT.**

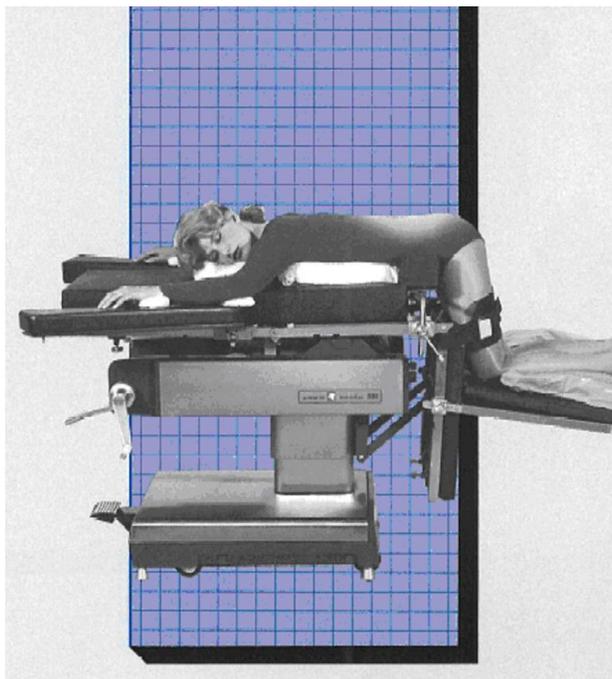


**LUMBAR LAMINECTOMY PATIENT IS SUPPORTED FOR LUMBAR LAMINECTOMY BY MORGAN PELVIC RESTS. THIS PREFERRED POSTURE SUPPORTS AND LEVELS THE LUMBAR AREA AND PROVIDES FREEDOM BENEATH THE ABDOMINAL AND CHEST AREA FOR IMPROVED RESPIRATION. PATIENT'S WEIGHT IS EVENLY DISTRIBUTED BETWEEN THE KNEES AND ILIAC CREST.**

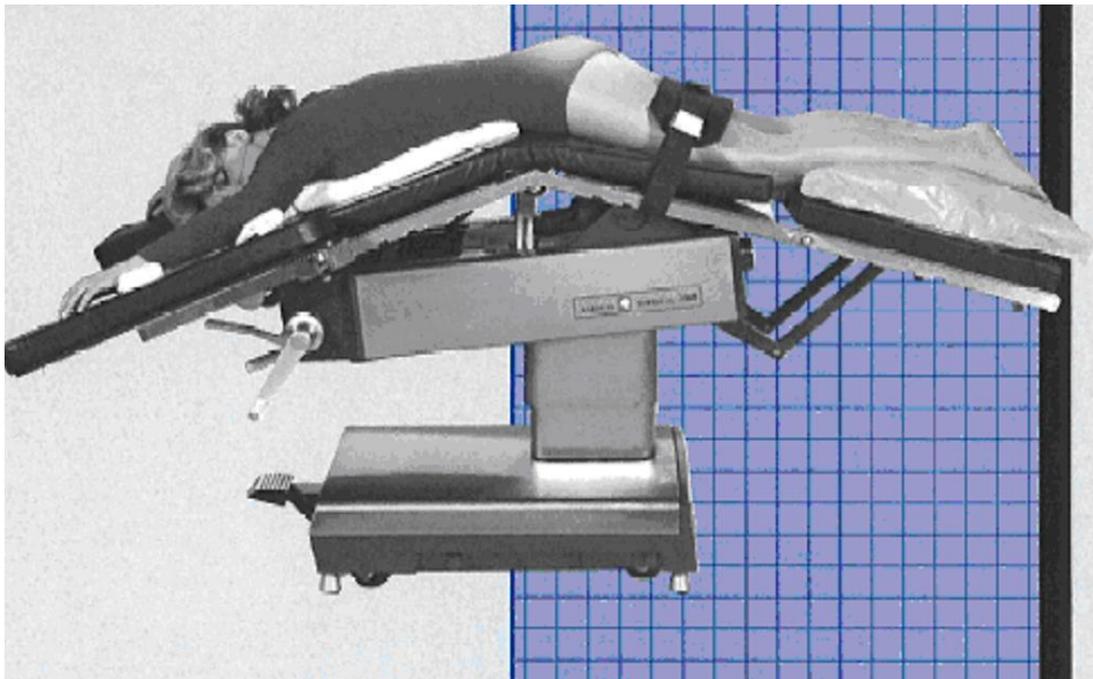


**PROCTOLOGY THE JACKKNIFE POSITION AIDS IN THE GRAVITATIONAL EXTENSION OF THE SIGMOID FLEXURE AND MINIMUM RESPIRATORY EMBARRASSMENT. MATTRESS PAD IS REVERSED TO COVER THE PERINEAL OPENING IN THE SEAT SECTION. MATTRESS PAD IS REVERSED TO COVER THE PERINEAL OPENING IN THE SEAT SECTION. FOOT EXTENSION AND HEAD SECTION ARE APPLIED TO THE SIDE RAILS OF THE FOOT SECTION AS INDICATED.**

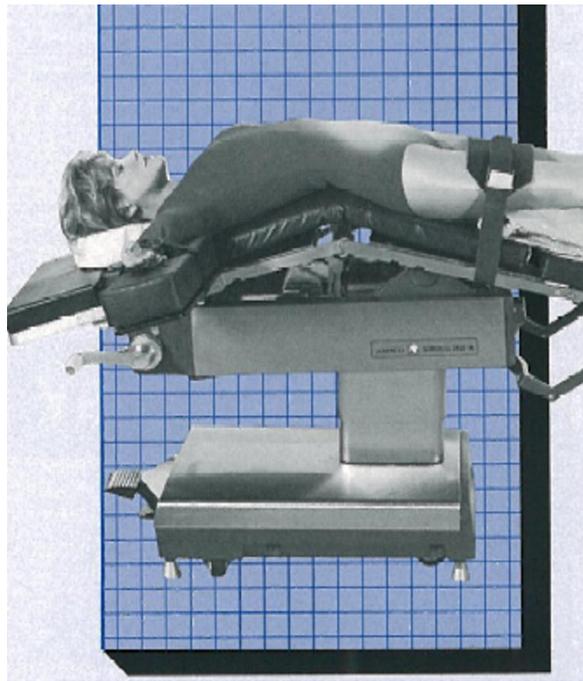


**GALL BLADDER FOR GALL BLADDER PROCEDURES, THE 2080 IS FLEXED, PUT INTO SLIGHT REVERSE TRENDELENBURG AND THE BODY ELEVATOR IS RETAINED AT EXACTLY THE SAME HEIGHT AS THE X-RAY TOP. THIS POSITION PROVIDES A CONSTANT CONTOUR AND SUPPORT AT THE TWELFTH THORACIC VERTEBRAE REGION. IN ADDITION TO EXCELLENT EXTENSION OF THE OPERATIVE AREA, THIS POSTURE GIVES FULL AND COMFORTABLE SUPPORT FOR THE PATIENT.**

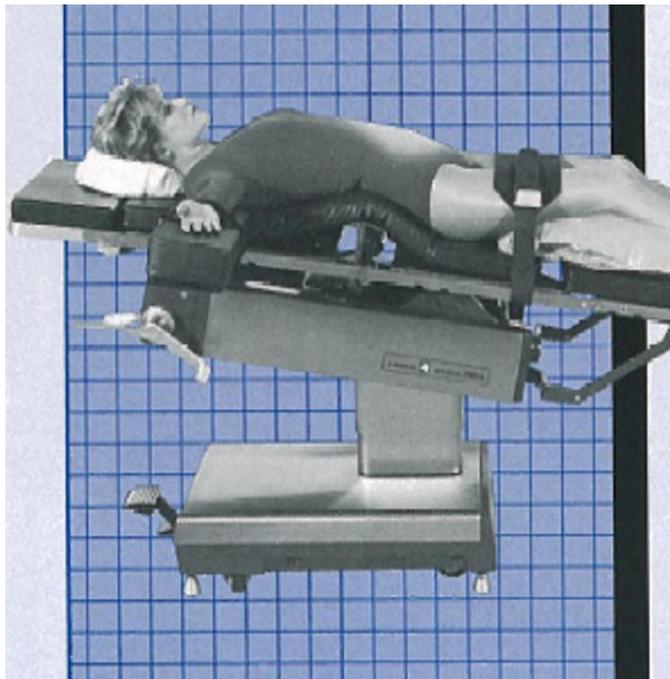


**GALL BLADDER FOR GALL BLADDER PROCEDURES, THE 2080 IS FLEXED, PUT INTO SLIGHT REVERSE TRENDELENBURG AND THE BODY ELEVATOR IS RETAINED AT EXACTLY THE SAME HEIGHT AS THE X-RAY TOP. THIS POSITION PROVIDES A CONSTANT CONTOUR AND SUPPORT AT THE TWELFTH THORACIC VERTEBRAE REGION. IN ADDITION TO EXCELLENT EXTENSION OF THE OPERATIVE AREA, THIS POSTURE GIVES FULL AND COMFORTABLE SUPPORT FOR THE PATIENT.**

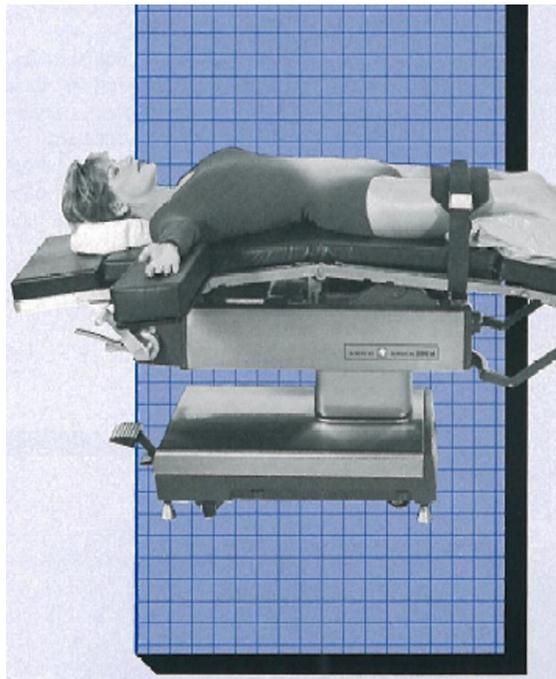


**ABDOMINAL THE GENERAL ABDOMINAL POSTURE ILLUSTRATED ON THE LEFT PROVIDES A MAXIMUM EXTENSION OF THE LEVEL ABDOMINAL CAVITY. 2080 IS FLEXED, FOOT SECTION LOWERED SLIGHTLY TO RELAX KNEES AND SLIGHT TRENDLENBURG LEVELS THE OPERATIVE SITE. INTRAOPERATIVE X-RAYS CAN BE TAKEN SIMPLY BY LEVELING THE TABLE. THE UNIVERSAL ANESTHESIA SCREEN SEPARATES STERILE AND NON-STERILE AREAS WITH EASY ANGULATION.**

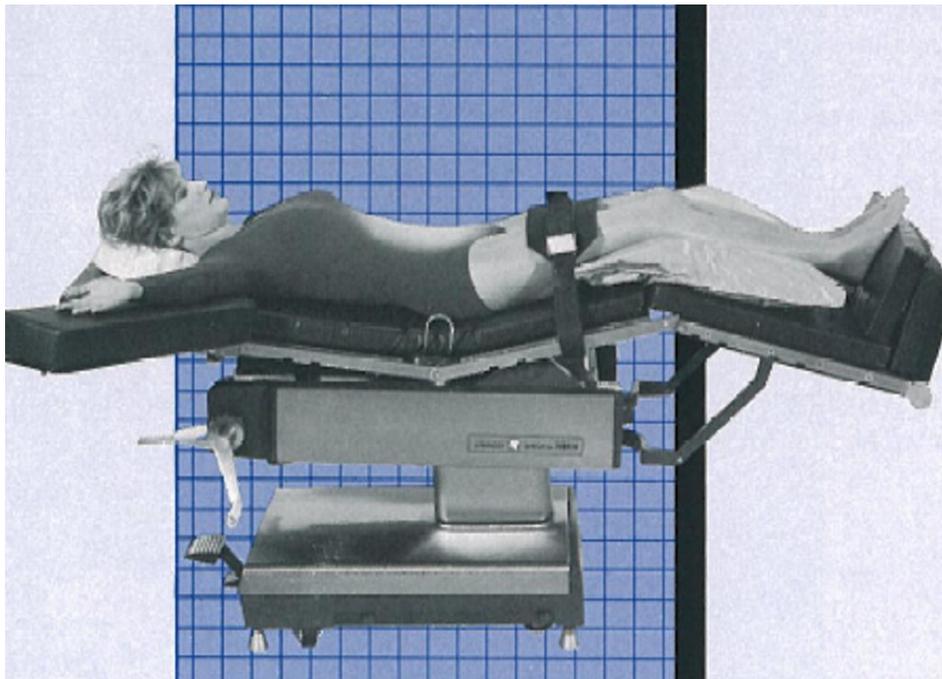

**ABDOMINAL THE GENERAL ABDOMINAL POSTURE ILLUSTRATED ON THE LEFT PROVIDES A MAXIMUM EXTENSION OF THE LEVEL ABDOMINAL CAVITY. 2080 IS FLEXED, FOOT SECTION LOWERED SLIGHTLY TO RELAX KNEES AND SLIGHT TRENDLENBURG LEVELS THE OPERATIVE SITE. INTRAOPERATIVE X-RAYS CAN BE TAKEN SIMPLY BY LEVELING THE TABLE. THE UNIVERSAL ANESTHESIA SCREEN SEPARATES STERILE AND NON-STERILE AREAS WITH EASY ANGULATION.**

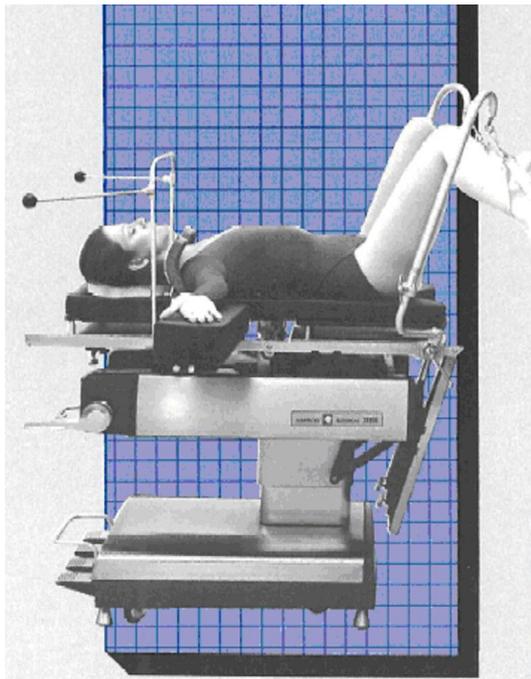

**GYNECOLOGY LITHOTOMY LEGHOLDERS ARE USED FOR HEMORRHOIDECTOMY, “D AND C” ANY PROCEDURE DURING WHICH LEGS MUST BE IN A COMFORTABLE POSITION AND A STABILIZED PELVIS IS NOT REQUIRED. FOOT SECTION IS FULLY RETRACTED TO ENABLE THE SURGEON TO SIT OR STAND COMFORTABLY CLOSE TO HIS PATIENT. FOOT EXTENSION MAY BE APPLIED FOR INSTRUMENTS OR FOR A GENERAL WORKING SHELF.**

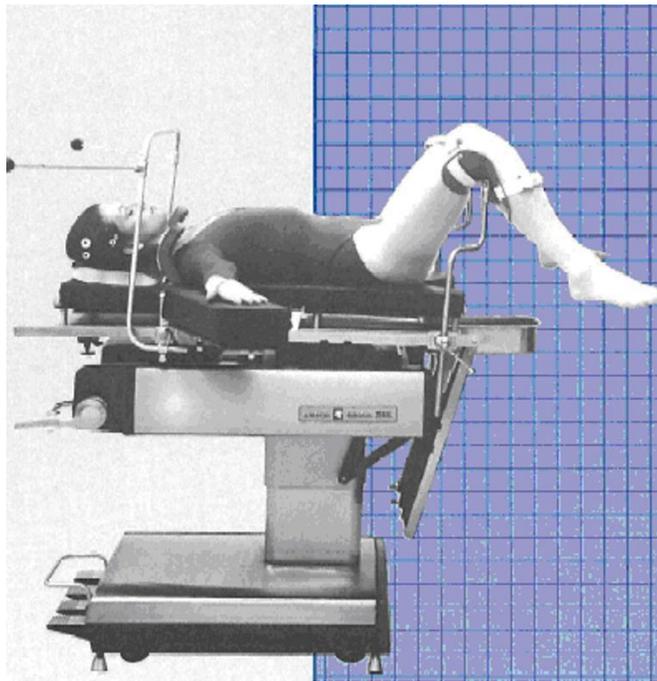

**CYSTOSCOPY FOR CYSTOSCOPY OR VAGINAL PREPPING WHEN USING LOCAL ANESTHETIC OR WHEN PATIENTS SHOULD HAVE CONTROL OF THEIR LEGS, BIERHOFF LEG CRUTCHES MAY BE UTILIZED. THE PERINEAL CUTOUT PERMITS APPLICATION OF THE HESS TRANSURETHRAL DRAIN TRAY. FULLY RETRACTED FOOT SECTION ALLOWS AMPLE SPACE FOR UROLOGIST'S OR SURGEON'S KNEES.**

**TABLE, OPERATING  
PHYSICAL & ENVIRONMENTAL SPECIFICATIONS**

<b>Item Dimensions</b>	<b>Height (Inches)</b>	<b>52</b>
	<b>Width (Inches)</b>	<b>32.5</b>
	<b>Length (Inches)</b>	<b>44.5</b>
	<b>Cube (Cubic Feet)</b>	<b>43.5214</b>
	<b>Weight</b>	<b>800 lbs</b>
<b>Ship Dimensions</b>	<b>Height (Inches)</b>	
	<b>Width (Inches)</b>	
	<b>Length (Inches)</b>	
	<b>Cube (Cubic Feet)</b>	
	<b>Weight</b>	
<b>Storage Temp</b>		
<b>Operating Temp</b>		
<b>Storage Humidity</b>		
<b>Operating Humidity</b>		
<b>Amps</b>		
<b>Watts</b>		
<b>Power Requirements</b>	N/A	
<b>Notes</b>		

**TABLE, OPERATING  
ACCESSORIES & CONSUMABLES - STARTUP**

NSN	Notes	Nomenclature	MFR PN Steris Corporatio n	DIST PN	QTY	UI	Unit Price	Total Price	Shelf Life Mon.
6530013539883		Table, Operating	2080 MIA		1	EA	\$21,680.73	\$21,680.73	N/A
	No NSN Required	Perineal Filler Kit	BF31-500		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6515011399416		Universal Anesthesia Screen	BF07-400		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6515011399424		Wristlets w/Straps & Brackets	BF11-100		1	PR	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6515014063927		Multipurpose Legholders	BF31-700		1	PR	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011399371		Conductive Restraint Strap, 66" Long	BF08-800		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011399411		Krause Arm Support	BF10-000		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011399417		Culdoscopic Knee Rest	BF06-600		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011399418		Clark Sockets (Pair)	BF08-300		1	PR	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011399419		Winged Anesthesia Screen	BF07-500		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011399425		Morgan Pelvic Rest	BF13-500		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011399426		Adhesive, Strap Holders	BF08-500		1	PR	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011399438		Transurethral Drainage Tray	BF10-600		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011399446		Locks for Siderail	BF13-300		1	PR	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011410770		Foot Extension w/Siderail Locks	BF03-100		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530011410772		Cassette Tray w/Handle	BF13-700		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530014064022		Tri-Clamps	BF31-200		1	PR	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530014067819		Support Arm w/2-inch Pad	BF37-000		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530014192287		Image Amplification Board	BF16-400		1	EA	Incl w/NSN 9883	Incl w/NSN 9883	N/A
6530014624962		Lithotomy Legholders	BF05-400		1	PR	Incl w/NSN 9883	Incl w/NSN 9883	N/A