

6515-01-453-4003
Defibrillator ECG Monitor/Recorder, LIFEPAK 10

[B-Before Operation, D-During Operation, A-After Operation, Q-Quarterly, and S-Semiannually]

ITEM NO	INTERVAL	ITEM TO BE INSPECTED AND PROCEDURE	IS NOT MISSION CAPABLE IF:
1	B, A	<p>Defibrillator</p> <p>a. Conduct inventory to ensure that the items listed on the Equipment Parts and Accessories List are on hand.</p> <p>b. Inspect case, cables, and connectors for function.</p> <p>c. Inspect defibrillator paddles for cleanliness and deep pits.</p> <p>d. Verify that the Medical Equipment Verification/Certification sticker (DD Form 2163) and the Defibrillator Energy Output Certificate (DA label 175) have current dates.</p>	<p>FASTPAK batteries (unless AC auxiliary power module is in use) or ECG 3-lead cable are missing or damaged.</p> <p>Damaged or inoperative components preclude operation.</p> <p>Paddles are dirty or pitted.</p> <p>The defibrillator or the defibrillator output has not been verified within the last six months.</p>
2	B	<p>Testing</p> <p>a. Monitor/Recorder</p> <p>Conduct the testing procedures as directed by the Operating Instructions.</p> <p>b. Defibrillator</p> <p>Conduct the testing procedures as directed by the Operating Instructions.</p> <p>c. Synchronizer Function</p> <p>Conduct the testing procedures as directed by the Operating Instructions.</p> <p>d. Quik-Pace Noninvasive Pacemaker</p> <p>(1) Conduct the testing procedures as directed by the Operating Instructions.</p> <p>(2) Inspect and test the pacing cable as directed by the Operating Instructions.</p> <p>e. Fast-Patch Adapter</p> <p>(1) Conduct the testing procedures using "Quick Test Cable Tester" as directed by the Operating Instructions.</p>	<p>Any of the monitor/recorder test procedures fail.</p> <p>Any of the defibrillator test procedures fail.</p> <p>Any of the synchronizer function test procedures fail.</p> <p>Any of the "Quik-Pace" noninvasive pacemaker test procedures fail.</p> <p>Any discrepancy is detected.</p> <p>Any of the Fast-Patch adapter test procedures fail.</p>

(continued) Appendix A. Operator PMCS

6515-01-453-4003
Defibrillator ECG Monitor/Recorder, LIFEPAK 10

[B-Before Operation, D-During Operation, A-After Operation, Q-Quarterly, and S-Semiannually]

ITEM NO	INTERVAL	ITEM TO BE INSPECTED AND PROCEDURE	IS NOT MISSION CAPABLE IF:
3	Q	<p>(2) Conduct the testing procedures using patient simulator as directed by the Operating Instructions.</p> <p>f. 12 Lead ECG Adapter</p> <p>Conduct the testing procedures as directed by the Operating Instructions</p> <p>Nickel-Cadmium Battery Maintenance</p> <p>a. Perform "Battery Reconditioning" test in accordance with Operating Instructions.</p> <p>b. Perform "Battery Shelf Life" test in accordance with Operating Instructions.</p>	<p>Any of the Fast-Patch adapter test procedures fail.</p> <p>Any of the 12 lead ECG adapter test procedures fail.</p> <p>The battery capacity is less than 80% after the third discharge.</p> <p>The battery capacity is less than 80% after the third discharge or the battery has more than 20% difference between the third and forth discharge.</p>
4	S	<p>Case</p> <p>a. Inspect for cracks, major dents, or puncture holes.</p> <p>b. Verify that the door cover closes and latches properly.</p>	