

6540-00-116-5780  
Edging Machine Ophthalmic Lens, Model Horizon II

[M-Monthly, Q-Quarterly, S-Semiannually, and A-Annually]

ITEM NO	INTERVAL	ITEM TO BE INSPECTED AND PROCEDURE	IS NOT MISSION CAPABLE IF:
1	A	<p><b>Edging Machine</b></p> <p>a. Conduct an inventory to ensure that the items listed on the Equipment Parts and Accessories List are on hand</p> <p>b. Inspect the unit for any damaged or excessively worn components.</p> <p>c. Be familiar with the control panel as directed by the Operators Preventive Maintenance Checks and Services.</p>	<p>Missing components or accessories prevent the operation of the edging machine.</p> <p>Damaged or deteriorated components prevent the operation of the edging machine.</p> <p>Being unfamiliar with the controls will prevent the operation of the edging machine.</p>
2	A	<p><b>Periodic Maintenance</b></p> <p>a. Verify that the daily, bi-weekly, monthly, and periodic preventive maintenance was performed as directed by the Operator Preventive Maintenance Checks and Services.</p> <p>b. Inspect the cutter motor brushes for wear as directed by the manufacturer's literature.</p> <p>c. Verify electrical safety.</p>	<p>The edging machine fails any of the electrical safety tests.</p>